CEEFIT

by SEAYER

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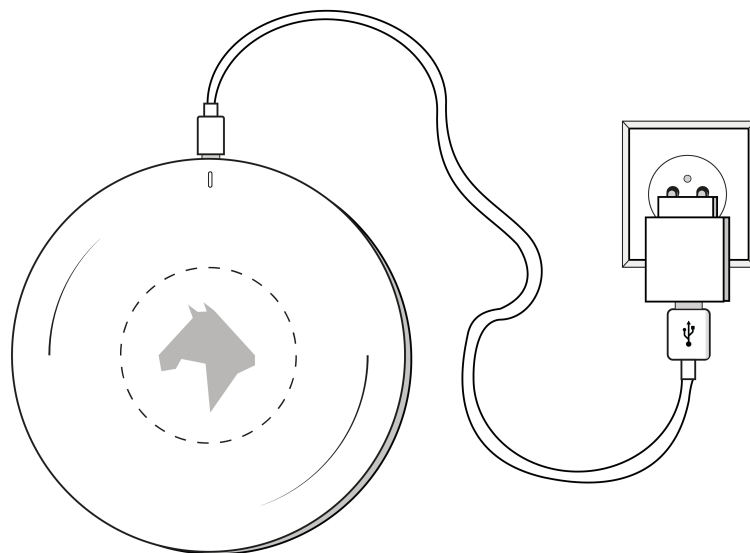
Plug the charger into the socket

Before you can use your CEEFIT, charge it with the charging platform supplied.

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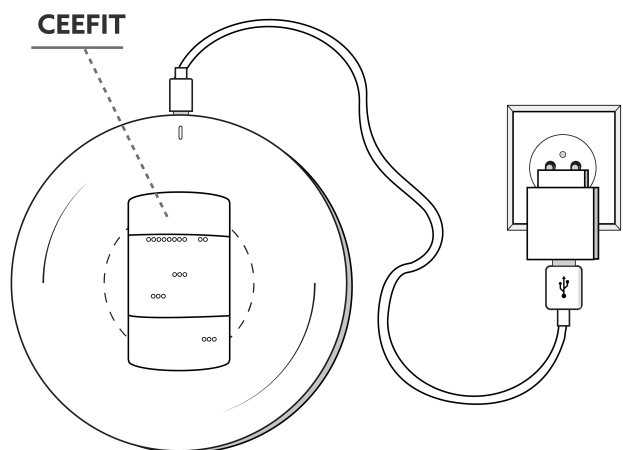
Position the CEEFIT casing

Place the CEEFIT casing onto the charging platform. The CEEFIT logo must face the logo on the platform. Also make sure to place the casing in the centre of the charging platform.

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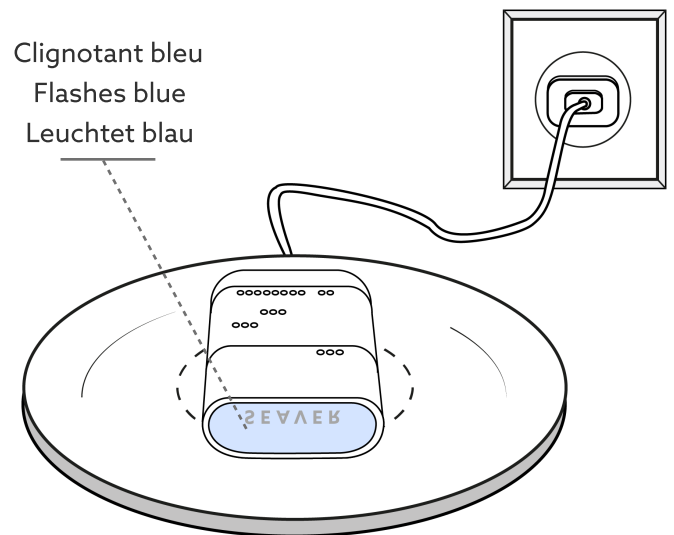
When charging

When the CEEFIT is charging, a blue light will switch on and flash inside the casing.

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Charging completed

The blue light will switch off once the charging is complete. It will take about 3.5/4 h for the sensor to be fully charged.

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Insert the sensor inside the silicone clip

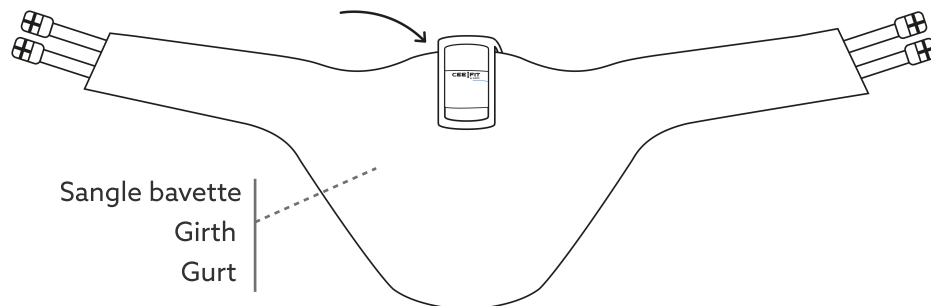
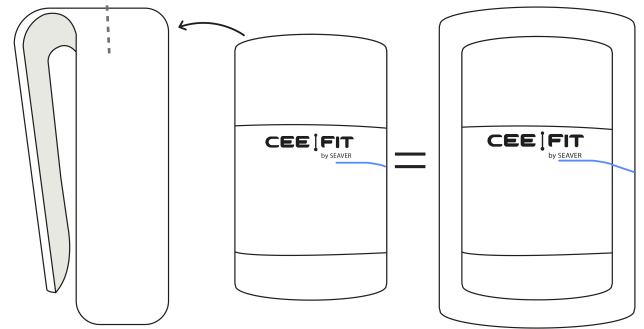
Insert the CEEFIT sensor inside the silicone clip and check that the blue light on the casing lines up with the one on the clip.

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Set up the CEEFIT on the girth

Make sure to place the clip in the centre of your girth.

If you use a training aid system and/or have a hook, move the clip slightly to one side.

For optimal functioning, we recommend a girth thickness of 8mm. Underneath, we advise you to use a padding (fabric, leather) under the clip.

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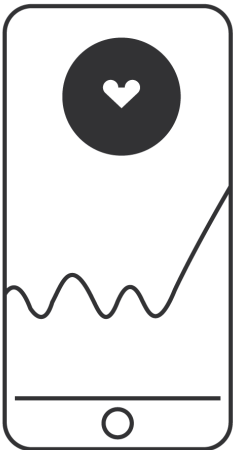
Connecting to the app

Download the Seaver app, create an account and then sign in.

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Download

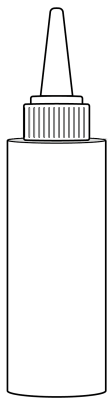
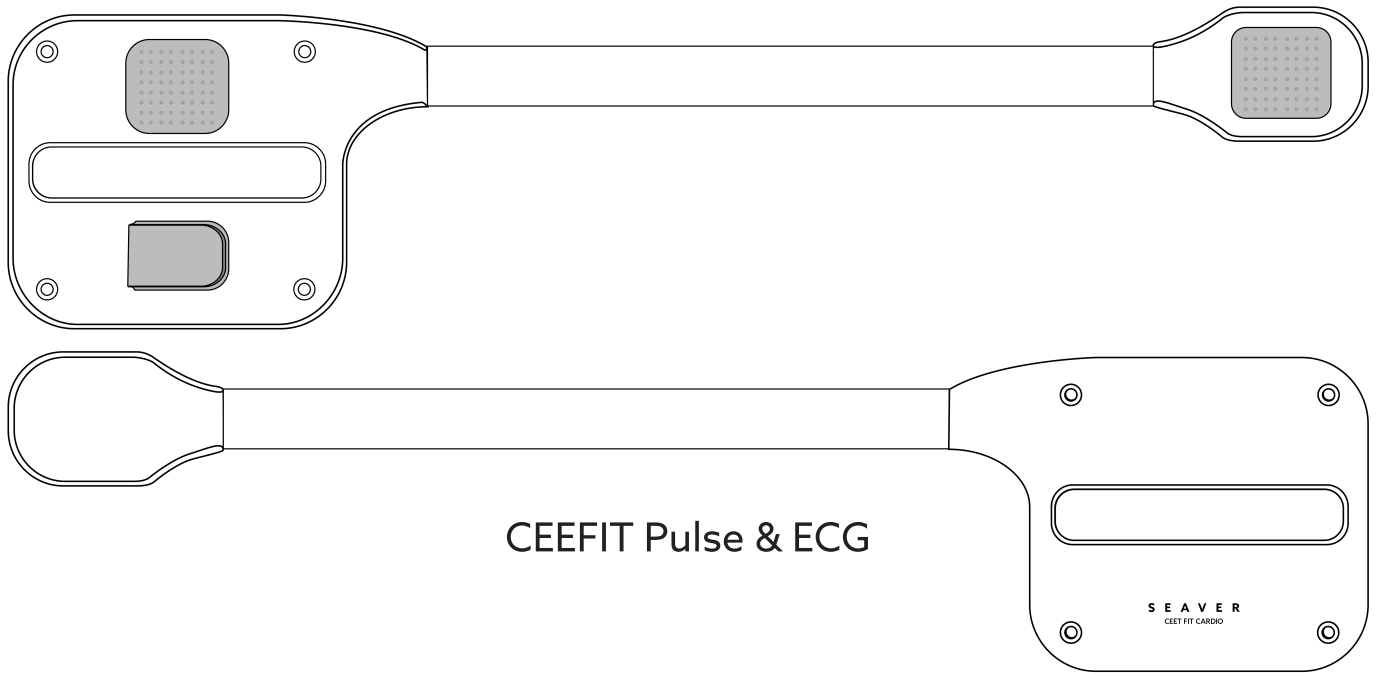
The Seaver app can be downloaded via standard platforms.

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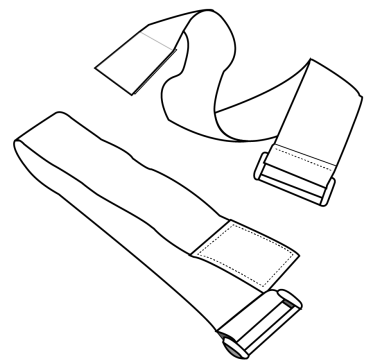
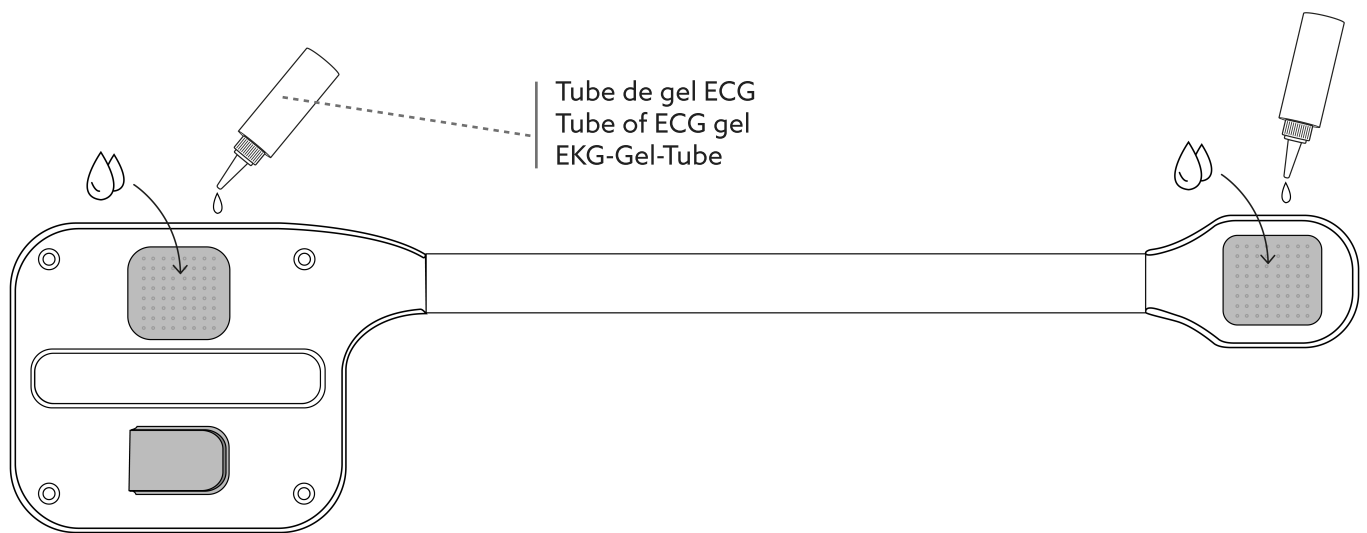
Apply some gel to the electrodes

Apply some gel to both electrodes before each use. You can also moisten them with water as well as where they rest on the horse.

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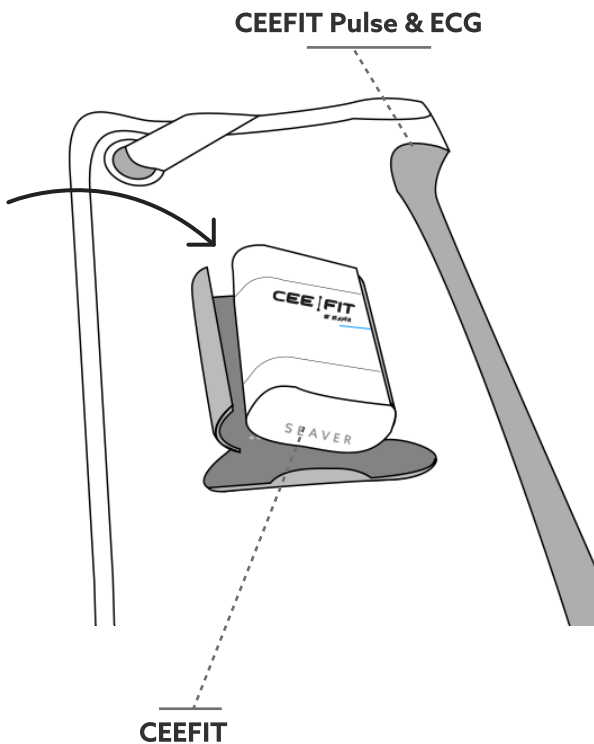
Fix the CEEFIT into the CEEFIT Pulse & ECG

- Clip the CEEFIT into the CEEFIT Pulse & ECG
- Make sure both blue lines line up
- A blue light flashes

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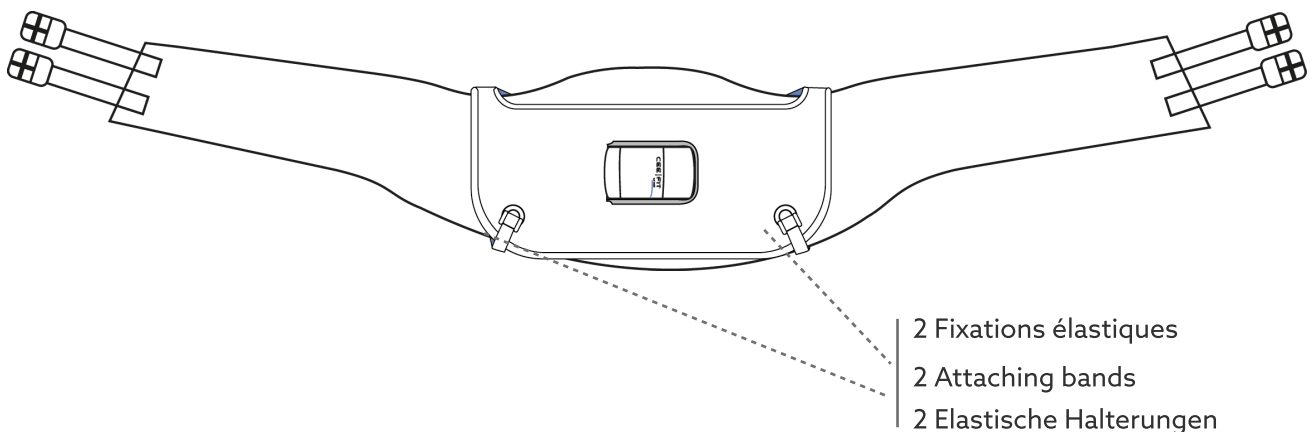
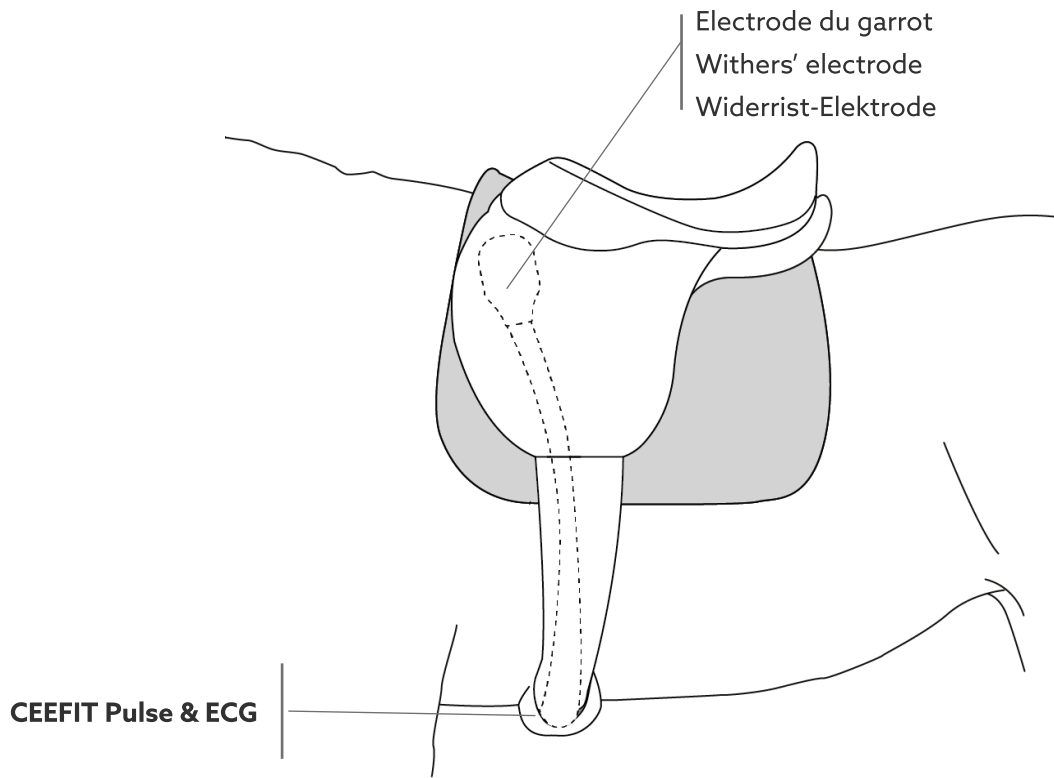
Fixing the equipment onto the girth

The CEEFIT Pulse & ECG needs to be well attached to the girth and centred so it does not move.

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Placing the withers' electrode

The electrode must be placed behind the withers on the left side of the horse, under the saddle pad. The electrode was scientifically tested. It does not apply any negative pressure on the horse's back.

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Useful to know :

1. Both speed and GPS data come from your phone or connected watch. It is recommended to keep one or the other with you while riding.
2. When riding indoors, the accuracy of GPS can be lower than usual.
3. Make sure not to unclip the CEEFIT from the CEEFIT Pulse & ECG before processing your training.
4. Jumping tracks are automatically detected when you jump more than 5 obstacles in a row at the same gait.
5. Jumps lower than 80 cm are not detected.
6. The height of your jump is equivalent to the distance from the floor to the device.
7. A symmetry grade is calculated when you trot for at least 10s on a straight line.
8. It is highly recommended to apply some gel to the electrodes. If you do not have gel with you, you can wet the electrodes with a sponge as well as the coat of the horse.

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If you have issues connecting the app with your ceefit please make sure that you:

1. Turn on Bluetooth on your phone
2. Turn on location services
3. Allow the app to pair with devices over Bluetooth
4. Do not pair your device outside the app
5. Allow the app to run in the background
6. Disable the 'energy saving' mode
7. With some Android smartphones, you will have to disable battery optimisation for our app (otherwise the system could kill the app during the training)
8. Enable Seaver to access location in your phone settings

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